

S2.0E Oil-Free Compressor



100% Oil Free | Dry Medical Air | Unlimited Hour Warranty
SKU: 2880110



Features:

- Oil-free
- Quiet operation with enclosure
- 100% duty cycle

Includes

- Compressor
- Tank
- Sound Enclosure
- Perma Dry Desiccant Dryer
- Pressure Filter Regulator
- Auto Condensate Drain
- 6' length of 1/4" Flex Line hose

Optional Add-Ons

- Buck Boost Part No: 2900601
- External Air Inlet Part No: 2880131

| | |
|------------------|--------------------------|
| Part # | 2880110 |
| Max (HV) Users | 4 |
| Operatories | 3-5 |
| Hp | 2 |
| kW | 1.5 |
| Amps | 9 |
| Volts (Min-Max) | 220V |
| Sound Level (dB) | 58 dB |
| Enclosure LxWxH | 29" x 25" x 32" |
| System Weight | 216 |
| Tank | 13 Gal, ASME & CRN Rated |
| CFM Max | 6.25 @ 80PSI |
| Operating PSI | 80-100 PSI |
| Dryer Type | Desiccant |

Site Requirements

| | |
|-------------------|-----------------------|
| Voltage (Min/Max) | 208 / 240V 1 60Hz |
| Full Load Amps | 9 |
| Breaker Rating | 20 Amps |
| Wire Size | 12 |
| Watts/Hour | 825 |
| BTU/Hour | 3000 |
| Install Footprint | 39" x 35" x 38" |
| Shipping Weight | 219 lbs. |

Note:

- Clean air intake manifold combines both compressor heads to a common 3/4" female copper fitting. For clean air intake from fresh air source use 1.5" - 2" PVC or ABS schedule 40 or better. Hospital and government clinics may require copper / stainless steel pipe (see NFPA or CSA requirements).
- Room must be provided with active ventilation. It's recommended to allow 18" around and above the compressor for easy access service.
- Room temperature should range 50 °F - 104 °F (10 °C - 40 °C).

1.800.668.8736 - sales@basevacdental.com - www.basevacdental.com